

Health Sciences Screening 2019

University of Otago







SURNAME	GIVEN NAMES	Lab no
ADDRESS		D.O.B.
		SEX

PAYMENT MUST BE MADE AT TIME OF COLLECTION

Photo ID must be sighted: Passport Driver's license Student ID card Other.....	\$282.00 (includes GST) Date of collect: Time of collect: Collected by: Collection Centre code:	<u>Samples required:</u> 2 x SST yellow top 1 x set of Quantiferon tubes
---	--	--

SITE SPECIFIC INSTRUCTIONS:

 <p> Doctor code: GCSTU Billing code: HSSM Panel codes: Profile =HSSM <i>Incubate & centrifuge QFG tubes</i> Forward all samples to SCL Dunedin </p>	 <p> Phlebotomy: **Bleed prior to 2pm Monday – Thursday only** <ul style="list-style-type: none"> Enter in CMS & charge Test = Otago UNIVERSITY Student Health Screen Transport in red bag Laboratory: Doctor code: GCSTU Billing code: HSSM Panel Code: QFG <ul style="list-style-type: none"> Incubate QFG: date & time samples Give form & SSTs to Immunology Immunology: centrifuge all tubes & forward to Sendaways Despatch all samples to Dunedin </p>	 <p> Do NOT barcode Do NOT add to Ultra <i>Incubate & centrifuge QFG tubes</i> Forward request form & all tubes to SCL Dunedin </p>
--	---	--

 <p> Collection Centre Instructions: <ul style="list-style-type: none"> If collection Centre does not do Quantiferon, refer patient to appropriate collection centre (Mon – Fri 7am – 1pm) Payment is required <ul style="list-style-type: none"> No additional collection fee to be added Centrifuge 2x SST gold tubes Send all specimens and form to Carbine Road </p>	<p> Data Entry Instructions: Doctor code: GCSTU Billing code: HSSM Panel code: =HSSM </p> <p> Specimen Services Instructions: <ul style="list-style-type: none"> Timestamp samples Photocopy form and scan to Scanviewer Give original form, 2x SST gold tubes & Quantiferon samples to Refers </p> <p> Refers Instructions: <ul style="list-style-type: none"> 2x SST tubes & original form to be held at 2-8°C until Quantiferon tubes are ready <ul style="list-style-type: none"> Place Quantiferon tubes in rack labelled 'Dunedin', at 8.00pm put rack in Biochem drop-off. </p> <p> Once Quant tubes are incubated & centrifuged, send all tubes (2x SST & Quant) with matched request form to SCL Dunedin </p>
--	---